

## Splitt tegninger

### Kilo







## 1. Hjulkrøss Kilo Integral




Pos.	Benevning	Ant.	Hepro Art.nr	HMS Art.nr	Stol
-	Hjulkrøss Kilo med integral	1	32201	-	30380/32550
1	Hjulkrøss Kilo 2020	1	32628	-	30380/32550
2	Trinse m/hjul Ø125 m/brems Kilo	2	32627	-	30380/32550
3	Trinse m/hjul Ø125 u/brems Kilo	2	32626	-	30380/32550
-	Hjulkrøss Kilo med integral	1	32203	-	32515
1.1	Hjulkrøss Kilo for Integral	1	32204	-	32515
2.1	Trinse Ø125 m/brems integral til Kilo	2	32748	-	32515
3.1	Trinse Ø125 u/brems integral til Kilo	2	32749	-	32515
4	Stang 6-kt. For brems, kromatis	1	30540	-	Alle
5	Bremsespike, komplett	1	32605	-	Alle
6	¼-fotring Hepro E1 Kilo	1	31397	-	Alle
7	EL.skap m/batteri, komplett	1	31403	-	Alle
8	Bryterboks kompl. m/kabel L120	1	31954	262441	Alle

*\*Være obs på hjulkrøss utformingen! Se hvilken som passer til din stol da denne varierer fra en eldre model (30380/32550) til nyere (32515)*

## 2. Løfteenhet

1.		2.		3.		4.	
							
Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.			
-	Løftesyule, LP2-2.1-100	1	61705	-			
1	Aktuator LA28.1.S-100-24.LP2	1	61693	-			
2	Løftesyule LP2-2	1	61704	-			
3	Gummidemper LP2	2	61786	-			
4	Festeskrus spindel/plate	8	61788	-			

### 3. Setebeslag

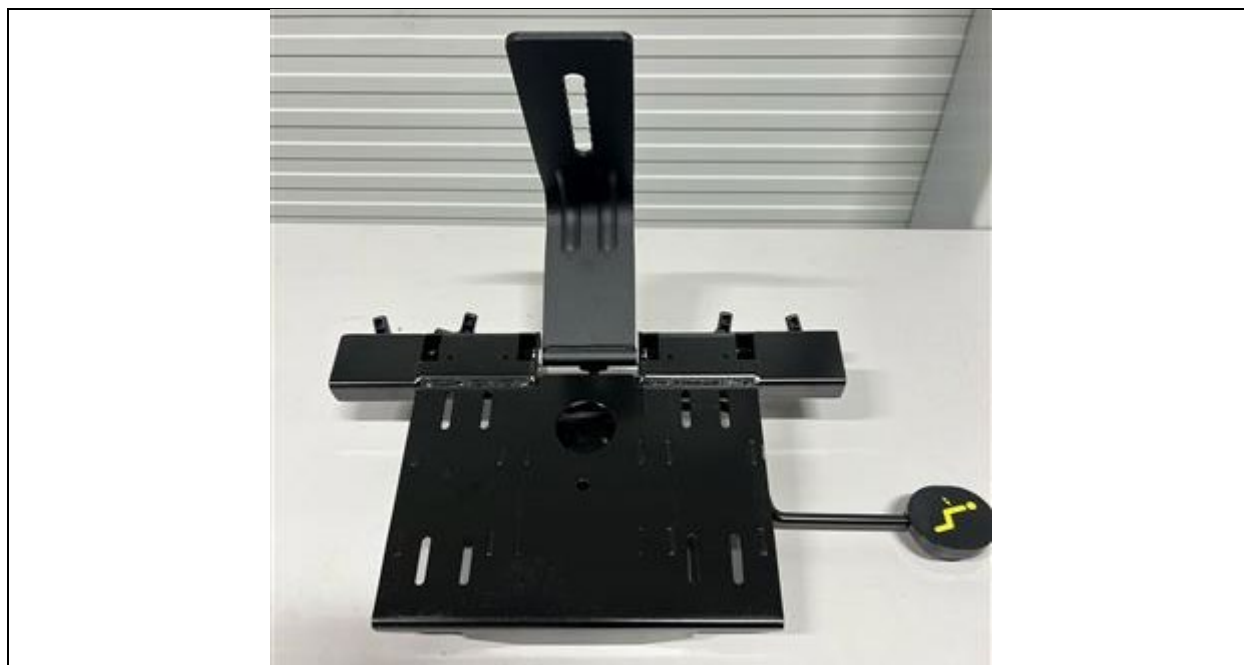


Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Setekryss Kilo 2020	1	32551	-

### 4. Armlene



Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Armlene Royal Høyre	1	31689	192230
-	Armlene Royal Venstre	1	31680	192231

### 5. Sete

				
Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Sete Kilo 55x50	1	32637	292209

### 6. Rygg

				
Pos.	Benevning	Antall	Hepro art. nr.	HMS art. nr.
-	Ryggpute Luna Stor	1	32636	292205