HEPRO

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
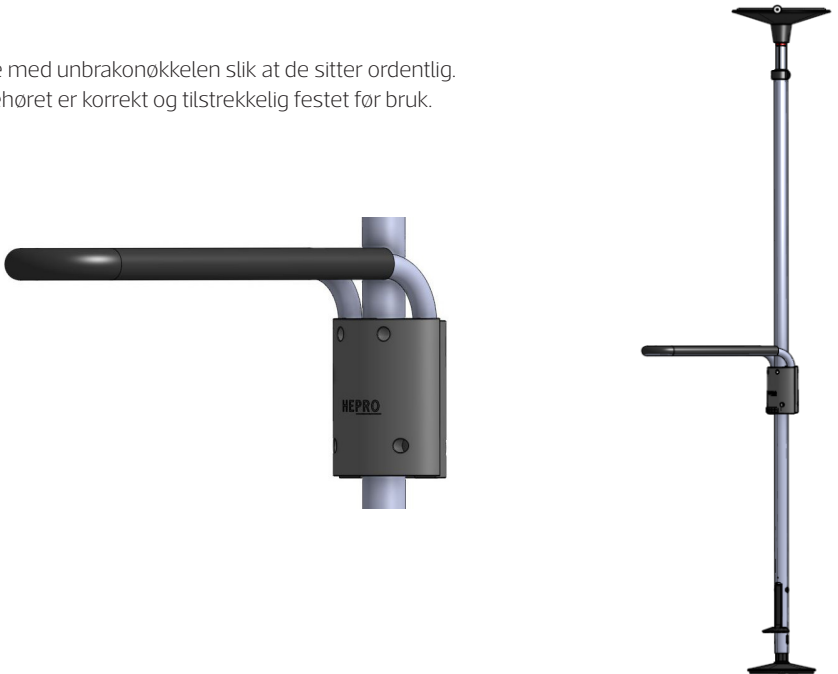
Assembling the Multi grip with rotation bracket Artnr. 35315


A

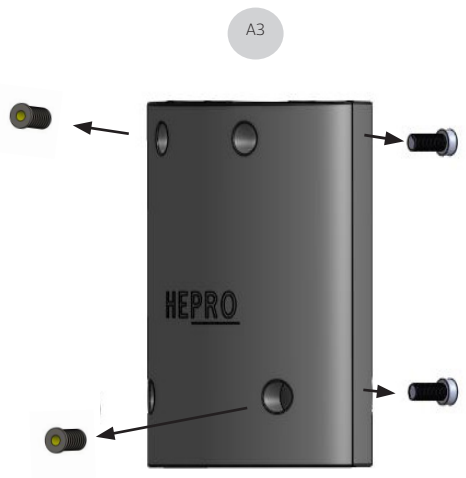The rotation bracket is delivered assembled and needs to be disassembled to be able to attach it to the Gripo. Start by loosening the screws on the outer cover **A3** and separate the two outer covers.

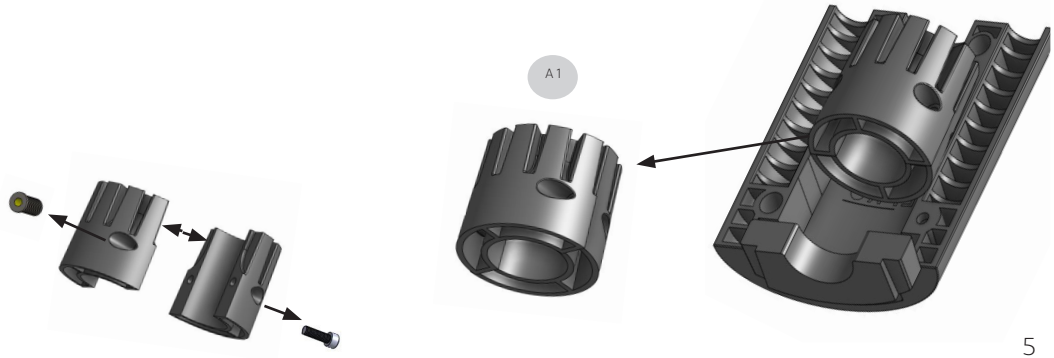Inside the outer cover **A3** is the inner gear sleeve **A1**. Take **A1** out of the outer cover and loosen the two screws that hold the inner gear sleeve together.


B

Start by attaching the two parts of the inner gear sleeve **A1** at the desired height on the Gripo pole. The inner gear sleeve is assembled using two screws and the hex key.


It is important that **A1** is assembled as shown in the illustration, with the gears pointing upwards.

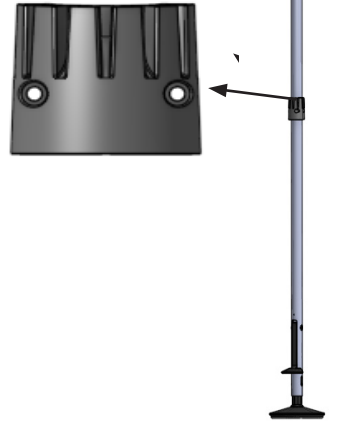Tighten the screws so the inner gear sleeve is properly attached to the Gripo.

C

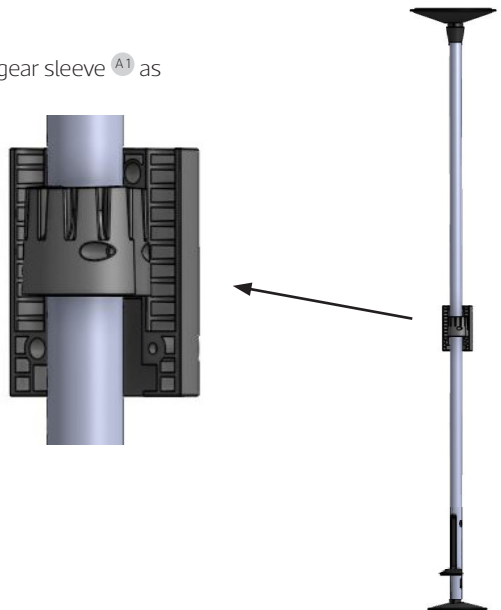
Thread one of the outer covers **A3** on to the inner gear sleeve **A1** as shown on the illustration.

D

Place the multigrip down in the outer cover ^{A3} that is threaded on to the inner gear sleeve ^{A1}. When the multi grip is in place, thread the second outer cover on to the inner gear sleeve.

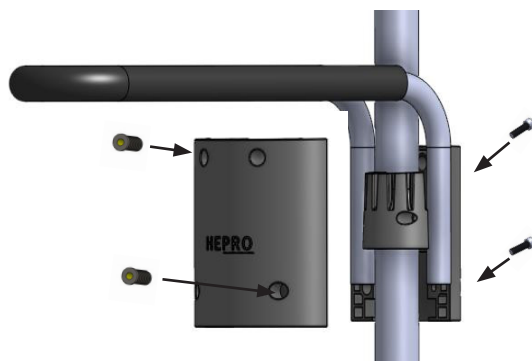
When the outer cover ^{A3} is complete, tighten the four screws using the hex key.

E

Tighten the screws properly.

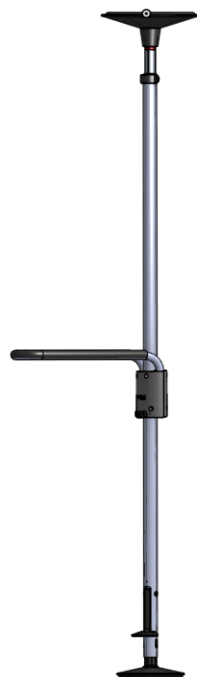Secure that the accessories are correctly fastened and safe before using it.

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