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Read **Installation Instructions** and **Instructions for Use** prior to the installation and use of **Bidette R3**. Keep the manuals for future reference.

## Bidette R3

Item no. 11 122

This product is approved and CE-marked in accordance with MDR 2017/745.


### Product Description

**Bidette R3** is an automatic bidet attachment with shower and drying function mounted on an existing toilet seat.

Shower jet and drying air are controlled by remote control.

**Bidette R3** can easily be programmed so that everyone can use it, regardless of needs and conditions.

**Bidette R3** is factory set with appropriate settings. Once installed, the bidet is immediately ready for use. All you need to do is set the water pressure and temperature on the mixer.

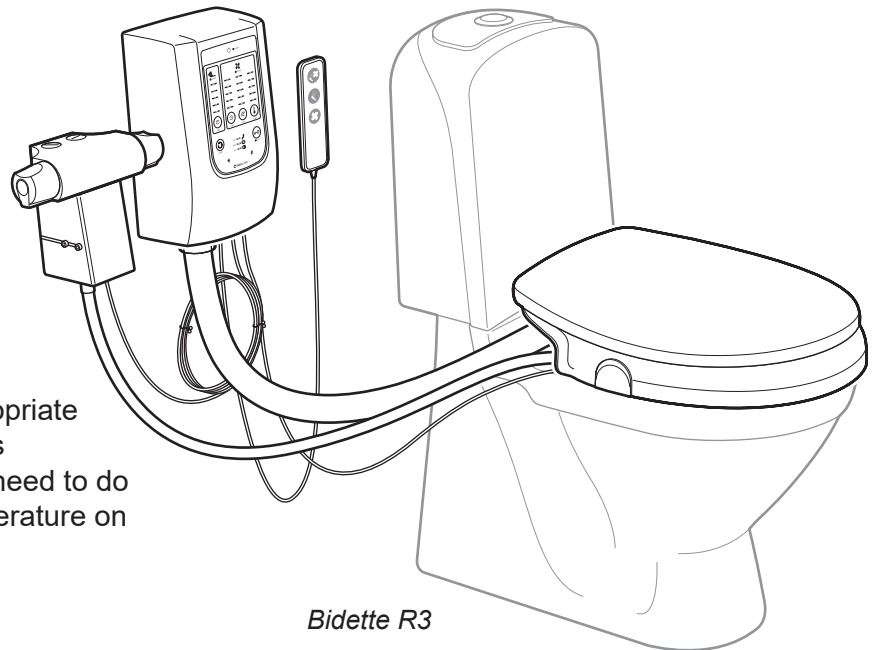*Bidette R3*

**Bruksanvisning**

sidorna 1 - 8

**Brugsanvisning**

siderne 17 - 24

**Bruksanvisning**

side 9 - 16

**User Manual**

pages 25 - 32

#### N.B.!

This product is not intended for use by persons (including children) with reduced physical/mental capabilities or lack of experience, unless they are supervised or instructed in the use of the product by a person responsible for their safety.

If there are doubts about the users movement patterns in relation to the capacity of the product, porcelain-mounted armrests,

#### N.B. / Technical Data:


wall-mounted armrests or other suitable means shall be used.

Children should be supervised to ensure that they do not play with this product.

**Maximum user weight on the seat: 380 kg**  
**Height of seat above toilet porcelain: 65 mm**  
**Estimated technical lifetime: 5 years**

# How Bidette R3 works

## Fully automatic

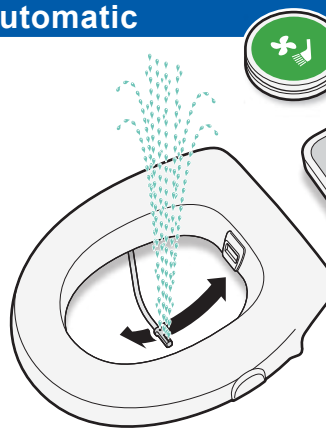
### Automatic button

A press of the green automatic button and Bidette R3:

You can always interrupt a step. With an extra push, you move on to the next step. If you use the bidet again within 15 minutes, the shower starts immediately without pre-flushing.

## Semi-automatic

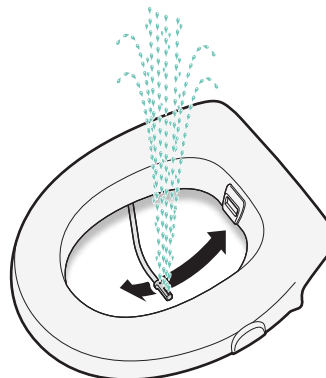
### Shower button

A press of the blue shower button and Bidette R3:

An extra push and the water is turned off. If you use the bidet again within 15 minutes, the shower starts immediately without pre-flushing.

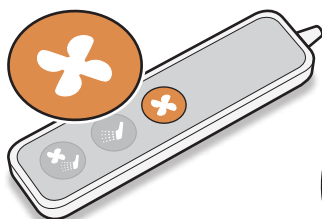
### Fan button

A press of the orange fan button and Bidette R3:

An extra push and the fan stops.

## Water Pressure and Water Temperature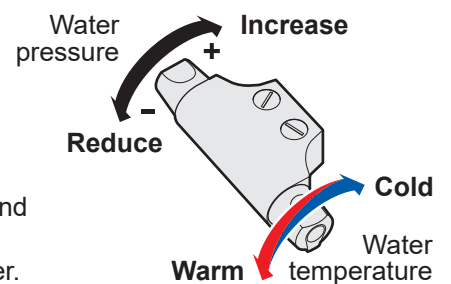

Set water pressure and water temperature of the thermostatic mixer.

**The bidet is now ready for use.**


If the bidet is not used for a protracted period, the mixer should be closed.

## Simpler settings for Bidette R3

### Change settings

1. Press **Power** so that the panel lights up.
2. Change your settings by pressing the respective **settings button**. The value is increased at each push of the button. After max, the value starts again from min.
3. **Ready!** The panel switches off after 15 minutes.


Bidette R3 is preset with appropriate values (the pictures show preset values). No other settings need to be made unless the user so prefers.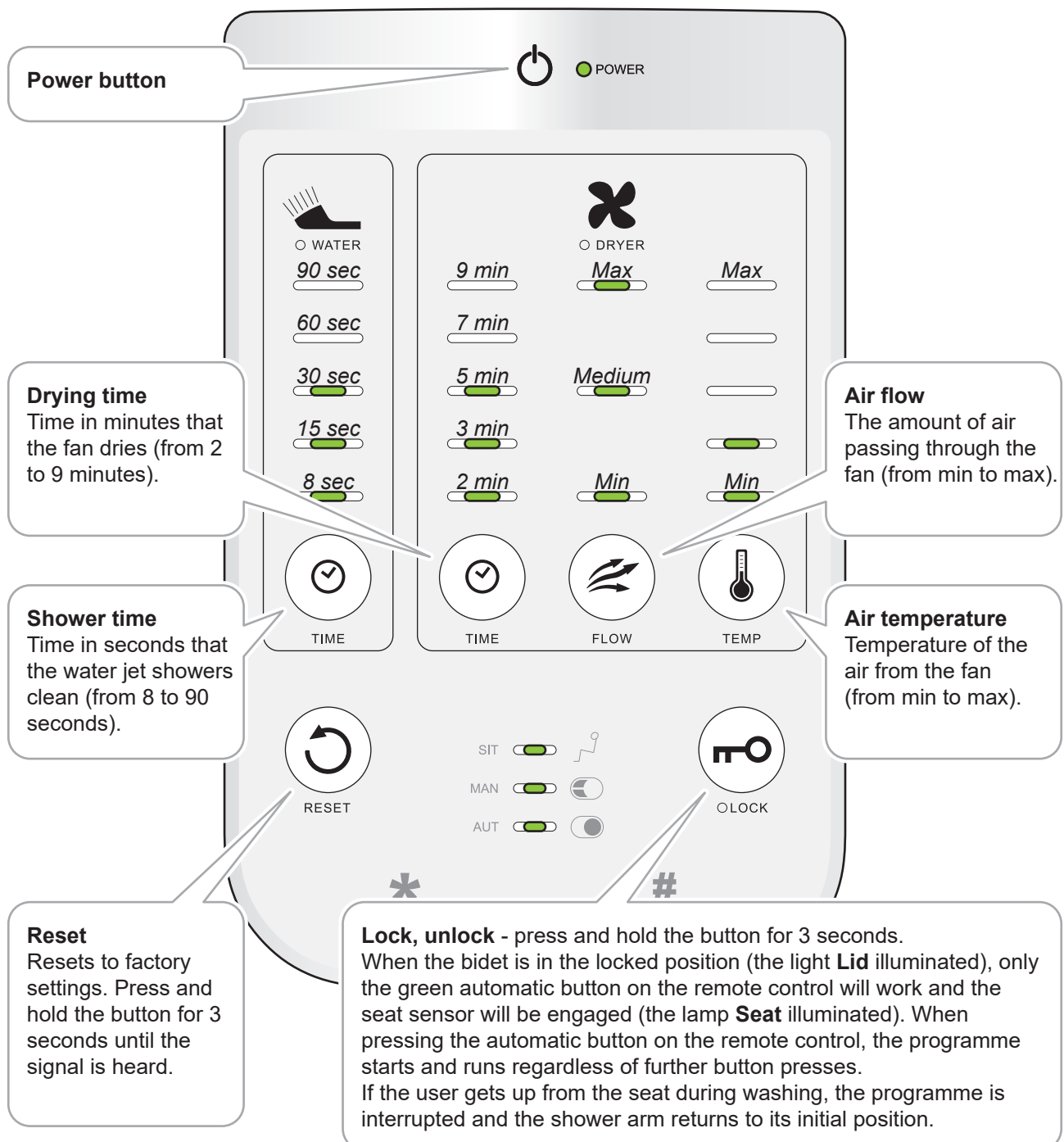

## Settings \*

### Change settings

1. Press **Power** so that the panel lights up.
2. Press **\*** for 3 seconds until the light **Water** begins to blink.
3. Change your settings.
4. Press **\*** to stop.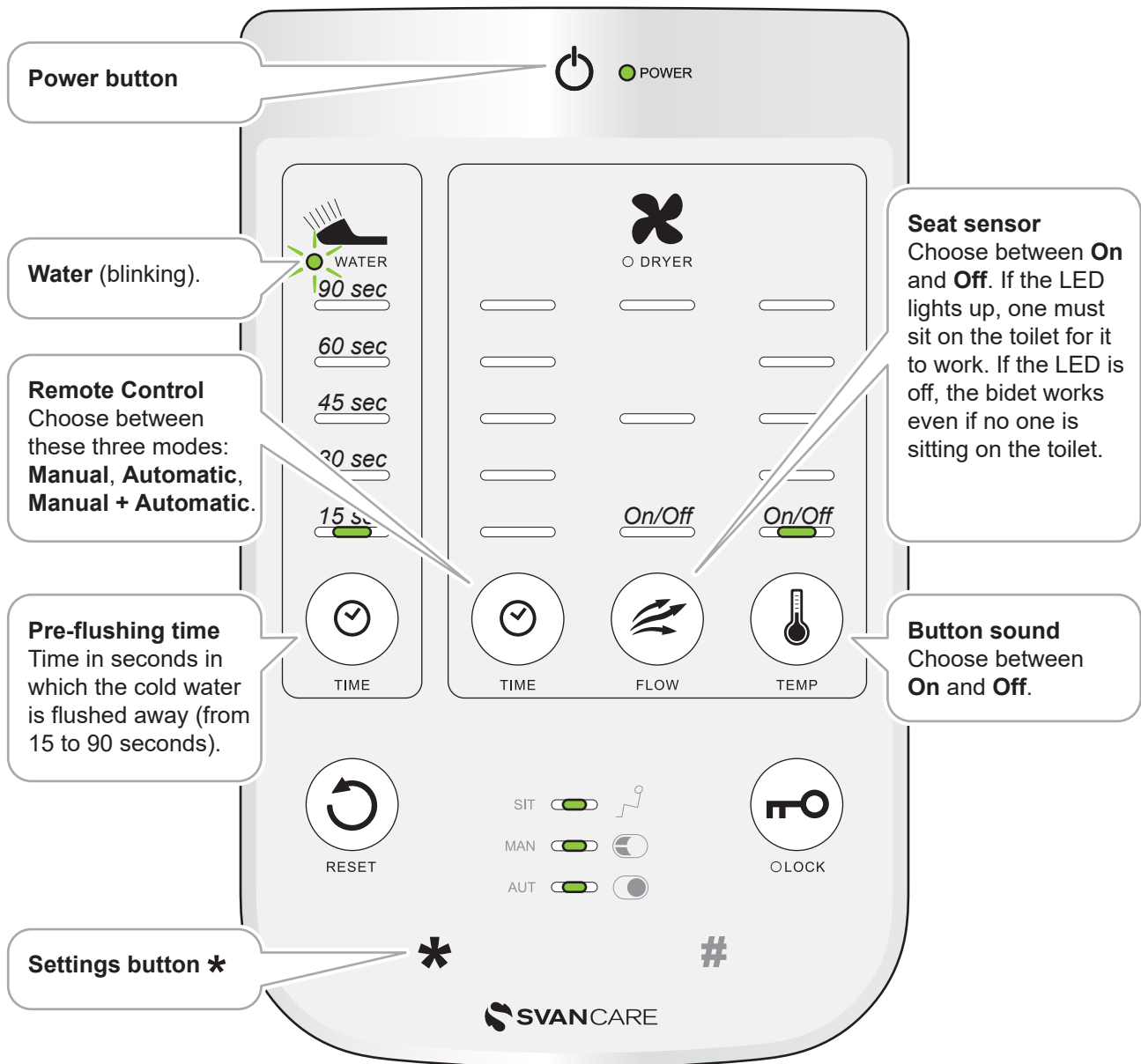


## Settings #

### Change settings

1. Press **Power** so that the panel lights up.
2. Press **#** for 3 seconds until the light **Dryer** begins to blink.
3. Change your settings
4. Press **#** to stop.

### The different positions of the shower arm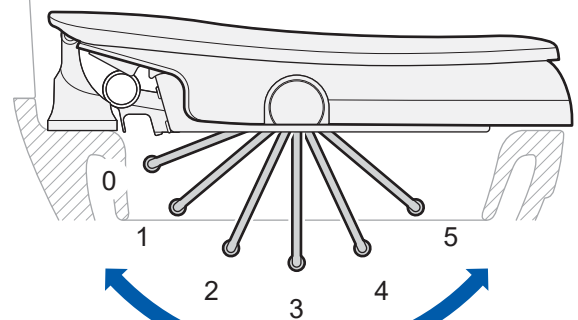

Shower arm's rear position (shower sends the water forwards).

Shower arm's forward position (shower sends the water backwards).

**Power button**

**Shower arm's forward turning position**  
Choose from six different positions (from position 0 to position 5). See sketch above.

**Shower arm's rear turning position**  
Choose from six different positions (from position 0 to position 5). See sketch above.

**Dryer (blinking).**

**Post-flushing time (3 sec.)**  
Choose between **On** and **Off**.

**Service position shower nozzle**  
Choose between **On** and **Off**. When servicing, the shower nozzle is positioned straight down (position 3). The fan also starts and runs for 3 seconds (with heat).

**Settings button #**

**POWER**

**WATER** (0-5)

**DRYER** (0-5)

**On/Off**

**TIME** (checkmark icon)

**FLOW** (fan icon)

**TEMP** (thermometer icon)

**RESET** (refresh icon)

**SIT** (person icon)

**MAN** (gear icon)


**AUT** (circle icon)

**OLOCK** (key icon)

**SVANCARE**

## Exploded Diagram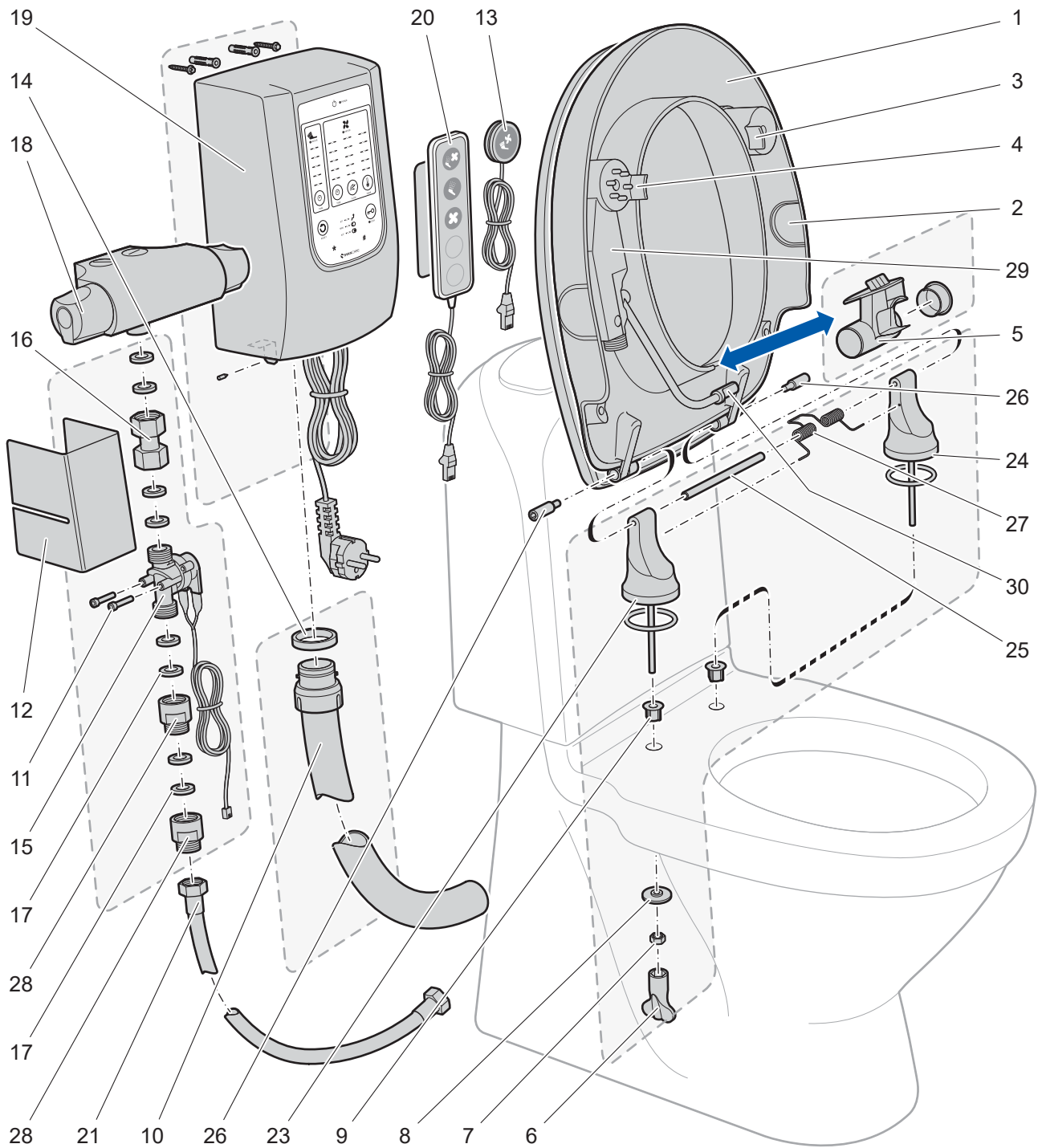

*Svan Care reserves the right to make changes to the component list.  
Current information can be found on our homepage.*

## Component list

No.	Ant.	Designation	Item number
1	1	Seat with lid, complete	11 251
2	2	Cover lid side	11 207
3	1	Lateral movement protection without hole	11 448
4	1	Lateral movement protection with hole	11 203
5	1	Air nozzle complete	11 226

No.	Ant.	Designation	Item number
6	2	Tool nut retainer	included in 11 289
7	2	Nut	included in 11 289
8	2	Washer	included in 11 289
9	2	Hole reducer porcelain	included in 11 289
10	1	Air hose complete (hose length 0.8 m)	11 331
11	2	Screw	included in 11 352
12	1	Valve cover	11 445
13	1	Remote control Single (cable length 1.35 m)	11 450
14	1	Gasket	included in 11 331
15	1	Solenoid valve complete (cable length 1.35 m)	11 352
16	1	Connection	included in 11 352
17	8	Gasket	included in 11 352
18	1	Thermostatic mixer	10 238
19	1	Fan housing R3/ M3 (cable length 1.5 m)	11 298
20	1	Remote control Triple, magnet, mounting plate (cable length 1.35 m)	11 466
21	1	High pressure hose (hose length 0.95 m)	11 334
23	1	Rear support leg with damper spring, complete	11 289
24	1	Support leg left	included in 11 289
25	1	Axel	included in 11 289
26	2	Locking screw	included in 11 289
27	1	Damper spring	included in 11 289
28	2	Rear valve R3/ M3	included in 11 352
29	1	Engine complete (cable length 1.9 m)	11 189
30	1	Shower nozzle adjustable	11 193

## Materials and recycling

Seat and cover lid are made of Urea-formaldehyde. The remote control is made of ABS plastic. Fan housing, remote control and solenoid valve are to be handed in as electrical waste, and all other waste as bulky waste.

## General tips and advice

Read the instructions carefully to learn how the product functions and how to set it up.

Bidette R3 can be easily programmed for use by anyone, regardless of one's needs and abilities.

If the user is a person with cognitive impairment, the lock function can help. See description of lock button on page 3.

If the person has difficulty getting up. Supplement the Bidette R3 with armrest or Lift.

A shower chair can be used over Bidette R3. This is thanks to the showerstream coming from below.

When using a shower chair, the seat sensor must be **Off**, see settings on page 4.

For new items and accessories for Bidette R3, please visit our website: [www.svancare.se](http://www.svancare.se)

## General cleaning

Wipe the fan housing and thermostatic mixer with a damp cloth.

**N.B.! Must not be rinsed off!**




**Bidette R3** and toilet are cleaned with a mild detergent that does not cause scratches.

## Cleaning the air hose and air nozzle

### Remove the air hose:

Loosen the locking screw (pos 1).

The air hose (pos 2) can now be removed, cleaned and rinsed separately.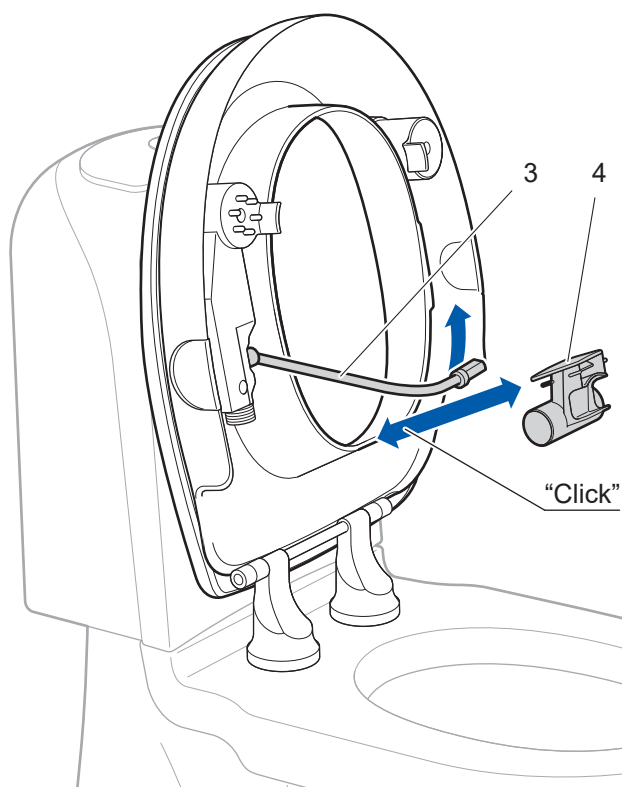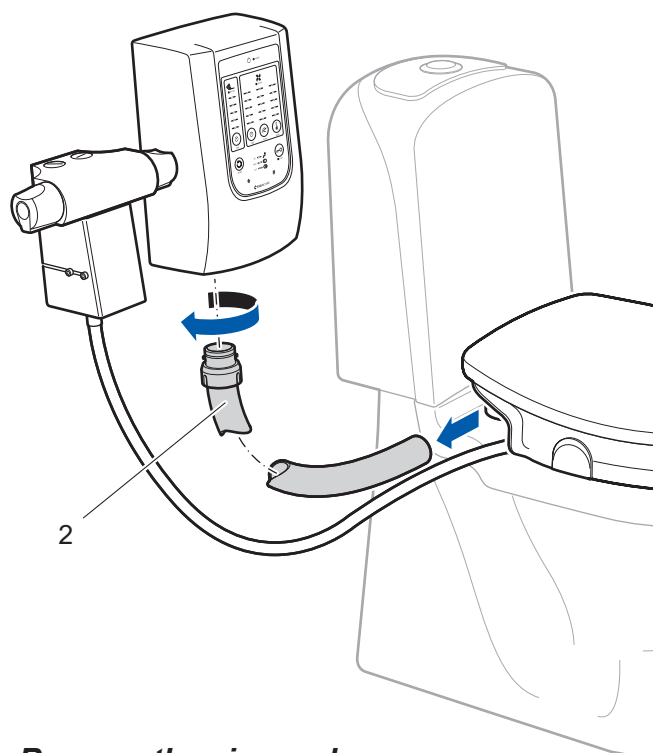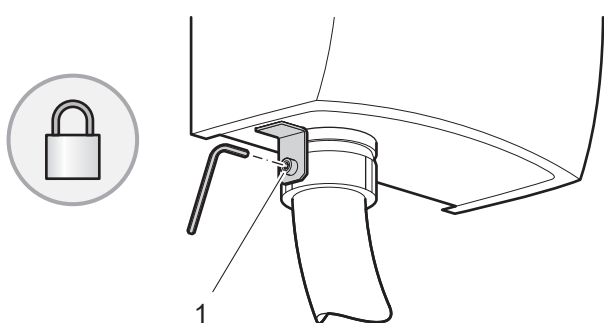

### Remove the air nozzle:

Fold up the seat.

Turn the shower arm (pos 3) so that it does not lock the air nozzle.

The air nozzle (pos 4) can now be detached from the seat, and also detached from the air hose, cleaned and rinsed separately.