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PRODUCENT

PDG Product Design Group
Unit 103 - 318 East Kent Ave. South
Vancouver, BC, Canada V5X 4N6

FORHANDLER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Danmark
Tlf.: +45 70 25 25 22
www.cobirehab.com

03/2019/JP

Instructions for using XXL-Rehab Manual wheelchair – The Eclipse Tilt

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 **To ensure proper use of the XXL-Rehab Eclipse Tilt, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .**

Also remember that in order to prevent the risk of injury to the user, the wheelchair may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Use

The XXL-Rehab Eclipse Tilt has a tilt mechanism which allows up to 30 degrees tilt for optimal comfort and positioning for individuals needing more support. The tilt uplift force is adjustable allowing the chair to be easily tilted with users weighing up to 270 kg.

The steel frame gives the XXL-Rehab Eclipse Tilt wheelchair maximum strength and rigidity, and therefore also makes it a long lasting wheelchair. The frame is designed so most of the user weight is carried on the rear wheels. The wheels are especially made for this wheelchair; they are extra hard and solid making propulsion and steering easy and with very little loss of energy.

The seat pan is solid. Various medical conditions demand a solid seat pan (fractures, hemiplegia, lateral shifts etc). Giving a bariatric user a solid seat pan facilitates a good posture and activeness in the chair.

The XXL-Rehab Eclipse Tilt wheelchair has adjustable seat depth, seat height, seat to floor angle and seat to back angle. All of this makes it easy to adjust to any user.

The back is constructed so users with a protruding behind can get good support to their lumbar spine.

Front wheels are placed in front of the seat pan to ensure an anterior anti-tip, which is very important taking into consideration that most bariatric users have a lot of weight placed in front of them. The anterior anti-tip makes it possible for the user to get in and out of the wheelchair without the risk of tilting.

 Using the product for individuals weighing more than 270 kg may pose a risk of tilting or damaging the wheelchair.

Special notes

 **WARNING/CAUTION** — notices as used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

Transporting a user secured in the wheelchair onboard a motorised vehicle.

Cobi Rehab do not recommend to transport wheelchair users secured in the XXL-Rehab Eclipse Tilt wheelchair onboard a motorised vehicle.

Component overview

1. Removable Armrest
2. Wheel Lock
3. Gas Strut Assembly (yellow spring)
4. Swing Away Front Rigging
5. Footrest
6. Front Caster
7. Tilt Activator (trigger)
8. Rear Wheel
9. Push Handle
10. Back Rest
11. Rear Axle Bolt
12. Lower Frame
13. Upper Seat Frame
14. Seat Cushion

Safety notes

Warnings

During your first use of the wheelchair, perform a number of exercises – together with your therapist or other health professional – in order to train your ability to safely reach, lean forward and transfer in or out of the wheelchair and become familiar with its safety limits.

- ⚠ **DO NOT ATTEMPT** to reach objects if you have to move forward in the seat.
- ⚠ **DO NOT ATTEMPT** to pick up objects from the floor if you have to reach down between your knees.
- ⚠ **DO NOT ATTEMPT** to reach for an object behind the wheelchair by leaning backwards over the backrest.
- ⚠ **DO NOT ATTEMPT** to shift your weight / sitting position in the direction of reach, as this COULD cause the wheelchair to tilt over.
- ⚠ **DO NOT** tilt the wheelchair without assistance.
- ⚠ **NEVER USE** an escalator to move the wheelchair between floors. Serious bodily injury may occur.
- ⚠ **DO NOT ATTEMPT** to stop a moving wheelchair with the wheel locks.
- ⚠ **ALWAYS ENGAGE THE LOCKS** on both wheels when transferring to or from the wheelchair.
- ⚠ **DO NOT** operate on roads, streets or highways.
- ⚠ **DO NOT ATTEMPT** to negotiate ramps or slopes steeper than 9°.
- ⚠ **DO NOT ATTEMPT** to negotiate ramps or slopes covered with ice, oil, or water.
- ⚠ **DO NOT ATTEMPT** to drive over obstacles, very uneven surfaces, etc. This could cause the wheelchair to tilt over, potentially causing personal injury.
- ⚠ **NEVER USE** unauthorized parts, accessories, etc., as this will void all warranties.
- ⚠ **NEVER LIFT** the wheelchair by any detachable parts (armrests, wheels, backrest, footrests, etc.).
- ⚠ **NEVER REMOVE** the anti-tippers.
- ⚠ **DO NOT USE** the footrests as a platform when getting in or out of the wheelchair.
- ⚠ **DO NOT ATTEMPT** to tilt the wheelchair without assistance.
- ⚠ **NEVER STAND** on the frame of the wheelchair.

Tire Pressure

If pneumatic tires are supplied, DO NOT use your wheelchair unless it has the proper tire pressure (p.s.i.). DO NOT over-inflate the tires. Failure to follow these suggestions may cause the tire to explode and cause bodily harm.

Replacement of a pneumatic tire or tube MUST be performed by an authorized dealer.

Weight training

The XXL-Rehab Eclipse Tilt wheelchair is not designed or tested for use during weight training. If the XXL-Rehab Eclipse Tilt wheelchair is used during weight training activities, Cobi Rehab shall not be liable for any personal injury whatsoever, and all warranties will immediately be voided.

Safe operation of the XXL-Rehab Eclipse Tilt wheelchair

Safe operation of the XXL-Rehab Eclipse Tilt wheelchair will require both you and your aide to be familiar with the instructions in this manual.

Together with your therapist, analyse the techniques required to get in and out of the wheelchair, reach for objects, etc. This will familiarise you with the “DOs” and “DON'Ts” of using the wheelchair. It is important to practice all transfer techniques until you feel completely safe, which will enable you to negotiate the most common architectural obstacles.

Stability and balance

To keep the wheelchair as stable as possible at all times, always maintain proper balance. The wheelchair is designed to stay upright during normal everyday activities, provided you do not move beyond the centre of gravity.

Almost all activities in a wheelchair will cause the centre of gravity to shift. Many activities will require you to reach, lean or transfer in and out of the wheelchair. These movements will cause a change to the normal balance, centre of gravity and weight distribution of the wheelchair. To establish your particular safety limits, practice all leaning, reaching and transfer activities with the assistance of a qualified aide/therapist before attempting active use of the wheelchair.

- ⚠ **DO NOT LEAN** further forward than the chair's armrests. Make sure that the casters are pointing forward when you lean forward. This can be ensured by first moving the chair slightly forward and then slightly backward in a straight line.

Coping with Everyday Obstacles

Coping with the irritation of everyday obstacles can be alleviated somewhat by learning how to manage your wheelchair. Keep in mind your center of gravity to maintain stability/balance.

A Note to Wheelchair Assistants

When assistance to the wheelchair user is required, remember to use good body mechanics. Keep your back straight and bend your knees whenever tilting the wheelchair or traversing curbs, or other impediments.

Be aware of any removable (detachable) parts. These must NEVER be used for hand-held or lifting supports, as they may be inadvertently released, resulting in possible injury to the user and/or assistant(s).

When learning a new assistance technique, have an experienced assistant help before attempting it alone.

Tilting

⚠ WARNING: DO NOT tilt the wheelchair without assistance.

When tilting the wheelchair, an assistant should grasp the back of the wheelchair on a non-removable (non-detachable) part. Inform the wheelchair occupant before tilting the wheelchair and remind him/her to lean back. Be sure the occupant's feet and hands are clear of all wheels.

Tilting – Curbs

After mastering the techniques of tilting the wheelchair, use this procedure to tackle shallow curbs, short stairs, etc.

Escalators

⚠ DO NOT use an escalator to move a wheelchair between floors. Serious bodily injury may occur.

FIG. 1

FIG. 1

Transferring to and from other seats

WARNING – BEFORE attempting to transfer in or out of the wheelchair, every precaution should be taken to reduce gap distance. Turn both casters toward the object you are transferring onto. Also be certain the wheel locks are engaged to help prevent wheels from moving.

⚠ CAUTION - When transferring, position yourself as far back as possible in the seat. This will prevent damaged upholstery and the possibility of the wheelchair tilting forward.

⚠ NOTE - This activity may be performed independently provided you have adequate mobility and upper body strength.

1. Position the wheelchair as close as possible along side the seat to which you are transferring, with the front casters pointing toward it. Engage wheel locks. Shift body weight into seat with transfer.
2. During independent transfer, little or no seat platform will be beneath you. Use a transfer board if at all possible.

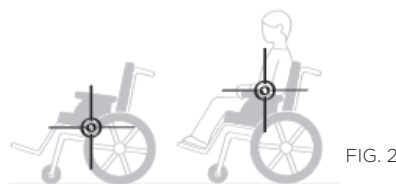
FIG. 2

FIG. 2

Change of weight distribution

⚠ WARNING - DO NOT attempt to reach objects if you have to move forward in the seat or pick them up from the floor by reaching down between your knees.

Many activities require the wheelchair owner to reach, bend and transfer in and out of the wheelchair. These movements will cause a change to the normal balance, the center of gravity, and the weight distribution of the wheelchair. To determine and establish your particular safety limits, practice bending, reaching and transferring activities in several combinations in the presence of a qualified health professional BEFORE attempting active use of the wheelchair.

FIG. 3

FIG. 3

Functional reach from a wheelchair

Proper positioning is essential for your safety. When reaching, leaning, bending forward, it is important to use the front casters as a tool to maintain stability and balance.

Unpacking and assembly

Unpacking

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify your dealer/Carrier immediately.
2. Remove all loose packing from the carton.
3. Carefully remove all components from the carton.

⚠ NOTE — Unless the XXL-Rehab Eclipse Tilt is to be assembled immediately, retain cartons and packing materials for use in storing the wheelchair until assembly is required.

Assembly

When unpacking XXL-Rehab Eclipse Tilt, you will need to assemble the armrests, backrest, front rigging (if applicable), connect the cable from the strut to the tilt activator (trigger) and anti-tip tubes to complete the set-up.

Backrest

The seat frame has two bolts installed along the left and right side of the rear of the seat frame. Remove these bolts on each side and place the backrest subassembly into the back of the seat frame and replace bolts and fasten securely. To attach tilt actuator cable to the back post (trigger) follow instructions shown on page 18.

Front Rigging

Installing the front rigging depends on the type of front rigging included with chair. For plug-in front rigging, push in the button on the side of the upper tube and slide into the frame until the button aligns with the hole on the frame and snaps into place. For the “swing-away” style front rigging, mount the front rigging on top of the already mounted adaptor and lock into place.

Arm Assembly

Install the arm assembly's by simply sliding the arms into the slotted arm receivers on the frame.

Anti-Tip Tubes

If anti-tip tubes are not already installed on the chair, locate them in the box and slide them into the receivers at the back underside of the frame so that the rollers are pointing downwards. These will prevent the chair from overturning if the centre of gravity moves too far rearward and are mandatory for safety and warranty protection.

⚠ IMPORTANT — With the user sitting in the chair, check to ensure the wheelchair is stable in all tilted/untilted positions before dispensing the wheelchair.

User customisation

The following instructions explain how to adjust the XXL-Rehab Eclipse Tilt wheelchair. After making adjustments, it is very important to check that all parts are properly secured.

Tools

Philips Screw Driver

Allen Keys: 1/8", 5/32", 3/16"

Adjustable or Open End Wrench: 7/16", 1-1/4", 1-5/16"

Socket Head Driver with socket Heads: 9/16", 3/4", 1-1/4"

Front rigging

This Procedure includes the following:

- Footrest Assembly Installation.
- Footrest Height Adjustment.

⚠ WARNING — After making adjustments, always make sure that parts are properly tightened BEFORE using the wheelchair.

Swing-Away Footrest Assembly Installation

1. To mount swing-away style front rigging, slide front rigging on already existing “swing-away adaptors”. The front rigging should ‘click’ and lock into place.
2. To release and swing away, push button on front rigging and push front rigging aside.

Footrest Height Adjustment

Height adjustment is made to the desired length with individual sitting in wheelchair with feet on footrests. The lowest point of the footrests should be no less than 2” from the ground or floor.

1. Remove the nut, bolt and coved washers and position the footrest assembly to a determined height.
2. Line up mounting hole and the aluminum insert in the footrest tube, reinsert the bolt and securely tighten by hand. Be sure to replace coved washers as well.
3. Repeat the procedure for the other footrest.

⚠ NOTE — Check to ensure that desired front rigging adjustment does not cause pressure to legs at front of seat upholstery. Such a condition could cause injury to the occupant.

⚠ WARNING

DO NOT ATTEMPT TO STAND ON FOOTRESTS. Standing on the footrests will cause the chair to tilt forward abruptly and may result in an injurious fall to the individual.

⚠ DO NOT ATTEMPT to lift chair by the footrests. These components are designed to separate from the wheelchair.

⚠ FRONT RIGGING SHOULD BE REMOVED when entering or exiting from the wheelchair.

Assembly

Armrests

This Procedure includes the following:

- Adjusting Armrest Height.
- Removing Armrests.
- Armrest Pads.

Adjusting Armrest Height (Dual Post Arms)

Unlock the arm by depressing the black release button on the upper front of the armrest while moving the armrest up or down. Adjust Armrest to desired height. The armrest height will 'lock' when the release button pops out of one of the armrest height adjustment holes.

Removing Armrests (Dual Post Arms)

Release the arm assembly by depressing the black release lever on the front of the armrest (just above the seat pan) while pulling the armrest up.

Adjusting Armrest Height (Single Post Arms)


Unlock the arm by depressing the black plastic release button on the side of the arm while moving the armrest up or down. Adjust Armrest to desired height. The armrest height will 'lock' when the release button pops out of the side of the arm.

Removing Armrests (Single Post Arms)

Release the arm assembly by lifting up on the black release lever of the arm receiver (mounted to the seat frame) while pulling the armrest up.

Armrest Pads

Check for rips or breaks. If these are present, replace immediately to maintain appearance and comfort. Armrest pad should sit flush against frame. If the pad is cracked or retaining screws are stripped, replace pad immediately.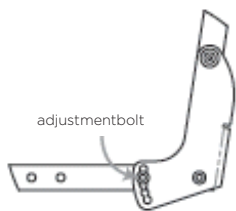


Seat and back

This Procedure includes the following:

- Adjusting the Back Angle.
- Adjusting the Seat Depth.

⚠ WARNING — After making adjustments, always make sure that parts are properly tightened BEFORE using the wheelchair.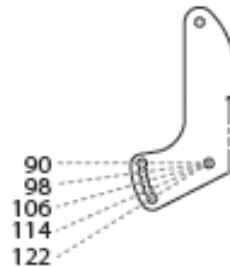

Adjusting the Back Angle

The seat-to-back angle may be set at a range of 90°-122°. Only 2 bolts need to be removed in order to change the angle.

1. To set the seat-to-back angle at 90°-122°, first loosen and remove the angle adjustment bolt shown. Note: there are two bolts; one on each side of the chair.
2. Adjust the bracket to the desired angle and re-install the bolts.
3. Retighten the nuts.

⚠ NOTE — The locknuts should be re-used no more than five (5) times to ensure locking material grips. If locknuts are worn, please replace.

Rear wheels

This Procedure includes the following:

- Removing/Installing the Rear Wheels
- Adjusting the Axle Location / Wheel Position Adjustment

Removing/Installing the Rear Wheels/Axles

1. Unless the product was ordered with quick-release axles, the rear wheels can be removed using a 1/2" socket head and a 1/2" crescent wrench. If the chair is configured using axle bolts (not quick-release), the axle bolt heads must be located on the inside of the frame with the nuts located on the outside of the rear wheel hubs. If axle bolts are installed backwards, the seat frame may hit the end of the axle bolts when attempting to tilt the chair. During assembly, tighten the axle bolts so that the rear wheels are snug but not so much that the wheels are unable to spin freely.
If quick-release axles have been supplied, push in the tip of the quick-release axle and pull the wheel off of the axle.
2. To reinstall the rear wheel onto the wheelchair, reverse step 1.

Assembly

⚠ WARNING. Make sure the detent pin and locking pins of the quick-release axle are fully released BEFORE operating the wheelchair. The locking pins MUST be protruding past the outside of the rear wheel axle receiver for a positive lock. Keep locking pins clean.

Adjusting the Rear Axle position


The rear wheel position may be adjusted forward or rearward.

1. There are six bolts on each of the axle plates which hold the axle plates tight onto the lower frame. To change the position of the rear wheels, loosen these bolts slightly, and slide the rear axles forward or rearward as desired. Before re-tightening, be sure both rear axles are in the same location as measured from the rear of the lower frame members. There are measuring indicators on the bottom of the lower frame members.
2. The antitip tubes slide into receivers at the back of the frame so that the rollers prevent the chair from overturning if the centre of gravity moves too far rearward.

⚠ IMPORTANT — With the user sitting in the chair, check to ensure the wheelchair is stable in all tilted/untitled positions before dispensing the wheelchair.

Setup of 16" / 22" / 24" Rear Wheel Anti-tippers
Axle positions are noted by the location of the rear edge of the axle plate on the ruled sticker on the frame.

The hole position on the anti-tip tube is shown.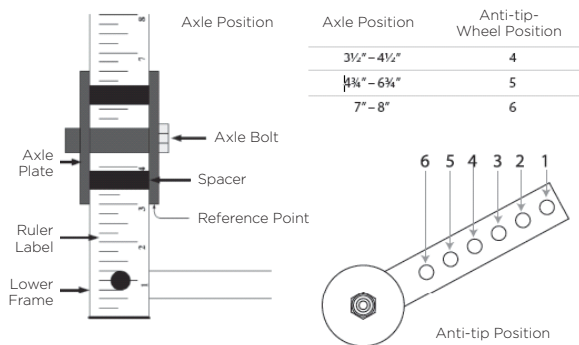

The extra positions on the anti-tipper (1 through 3) can be used to make the chair more stable, but it will reduce the caster clearance when tilted backwards.

Front casters

This Procedure Includes the Following:

- Installing/Replacing Front Casters

⚠ WARNING — After making Adjustments, always make sure that parts are properly tightened BEFORE using the wheelchair.


1. Remove bolt from front of caster housing.
2. Slide out caster.
3. Reassemble by reversing steps 1-2.
4. Test the wheelchair for maneuverability.

Seat-to-floor height

This Procedure includes the following:

- Adjusting Seat to Floor Height.

⚠ WARNING — After making Adjustments, always make sure that parts are properly replaced and tightened BEFORE using the wheelchair.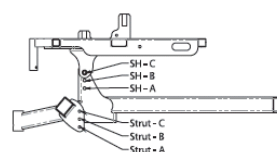

The seat-to-floor height on the XXL-Rehab Eclipse Tilt can be adjusted from 16" to 18".

1. Remove the pivot bolts that attach the upper seat frame to the lower base frame, these are two socket head bolts located on the outside of the seat pan assembly (one screw on each side).
2. Reposition the Seat Frame to the desired elevation. There are three elevation choices.
3. Remove the socket head bolt located on the Base Frame, which connects the lower end of the gas strut assembly to the Base Frame.
4. Reposition the lower end of the Gas Strut Assembly (Strut-A, B or C) to correspond to the selected seat elevation (SH-A, B or C) as shown.
5. Test the wheelchair to ensure the seat pan can be positioned approximately level when the gas strut is fully extended. Check to ensure all screws and nuts are assembled securely.

Adjustable seat tilt

This Procedure includes the following:

- Adjusting seat tilt.


⚠ WARNING— After making Adjustments, always make sure that parts are properly tightened BEFORE using the wheelchair.

1. To adjust the tilt angle, activate the tilting lever located on side of push handle or on the armrest, depending on how the wheelchair has been configured. Allow seat back to lower to desired level and release tilt lever. Seat should 'lock' at desired tilt angle.

Assembly


- Note that the tilt lever may be mounted onto the left or right side of the backrest push bar or on the left or right armrest.

⚠ WARNING — When tilting chair, attendant must hold push handle securely so that seat does not drop too quickly.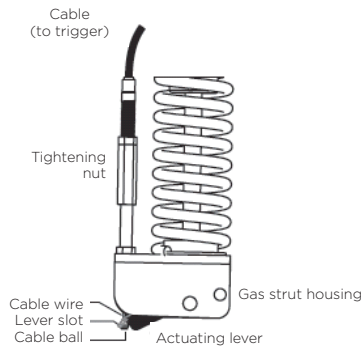


- To install the new cable, refit to the strut by reversing step 2. Be sure to connect the cable ball to the actuating lever.
- Re-fit the metal cable end by inserting into the side of the trigger.
- Feed the cable through to the front of the trigger.
- Hold the plastic parts in place and press the trigger, the cable should snap in place, if not you may need to push it until it snaps.


1 Fit metal cable-end into hole on trigger.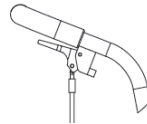

2 Feed cable through to the front of the trigger.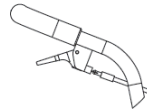

3 Push plastic parts together -><- until they "click" in place

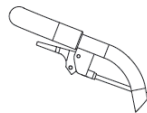4 Clip cable onto back post using plastic clips provided.

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Adjustable gas strut and cable assembly

Adjusting Cable Tension

Once receiving and assembling the chair, you may find that the chair tilts with user in it when the trigger is not actuated or chair will not tilt when trigger is actuated. This problem may also occur over time as the cable wire 'stretches'.

A simple procedure is needed to adjust the tension of the cable wire:

⚠ IMPORTANT — Before adjusting - be sure the steel ball (at bottom end of cable wire) is inserted into actuating lever slot and cable is properly attached to trigger.

- If the chair does not tilt when trigger is actuated - the cable wire is too loose and needs to be tightened. To do so, remove the rubber sleeve above the gas strut housing on the cable and turn the 'tightening nut' so less thread is showing above the nut.
- If the chair tilts without actuation - the cable tension is too tight and needs loosening. Reverse the procedure from above, making more thread showing above the 'tightening nut'.

Removing and Installing the Gas Strut Actuator Cable

The procedure is outlined below for cable replacement.

- Grab the cable just in front of the trigger. In one smooth motion, pull straight down until the cable releases from the end of the trigger. Pull the metal cable-end out from the side of the trigger.
- Disconnect the opposite end of the cable from the strut, using a 7/16" and a 3/8" open end wrench.

Tilt limiter


This Procedure includes the following:

- Adjusting the Tilt Limiter.

⚠ WARNING — After making Adjustments, always make sure that parts are properly tightened BEFORE using the wheelchair.

The tilt limiter is a yellow telescoping tube that is set beside the gas strut. It has 6 holes to set the tilt stop.

- Insert the silver 10-32 Socket Cap Screw in the desired position. Refer to section below for tilt degree settings.

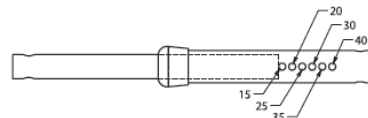Maintenance and cleaning

General notes

Cleaning

Regular cleaning of all surfaces will preserve the wheelchair's appearance and extend its service life. All surfaces may be cleaned with a mild soap solution. If required, spray the chair with VIRKON and wipe the fabric according to the instructions for use.

⚠ NEVER USE ANY SOLVENTS. You can also clean the chair by wiping it with spirit.

⚠ NOTE: washing and soaking the backrest upholstery will reduce the flame retarding effect.

Maintenance procedure

1. Check that all nuts and bolts are tight before using the XXL-Rehab Eclipse Tilt wheelchair. Check all parts for wear and damage, and replace parts where required. Check that the wheelchair is properly adjusted to fit the user.
2. Keep the axles free from dirt and fluff to ensure that the mechanism locks correctly and that the wheels can turn freely.
3. Remember to lubricate the axles at least once a month (use a 3-in-1 oil or a similar product).
⚠ WARNING — If pneumatic tires are used, do not use the wheelchair unless it has the proper tire pressure (p.s.i.). DO NOT over inflate the tires. Failure to follow these suggestions may cause the tire to explode and cause bodily harm.
4. If tires are pneumatic, recommended tire pressure is listed on the side wall of the tire.
5. Periodically check the wheels and tyres for cracks and wear, and replace as required.
6. Periodically check that the handrims are securely attached to the wheel.

In case of doubt, you can always contact the supplier.

In certain cases the XXL-Rehab Eclipse Tilt wheelchair must be inspected by the supplier. This applies especially in case of:

- cracks or damage to the frame.
- cracks or damage to a welding.

Always contact the supplier before returning a XXL-Rehab Eclipse Tilt wheelchair or any part of it. When returning the wheelchair, it is important to pack it in a box which is secured to the pallet. Cobi Rehab does not accept liability for transport damage to items returned to Cobi Rehab.

⚠ NOTE — Twice a year take your wheelchair to the dealer for a thorough inspection and servicing.

Regular cleaning will reveal loose or worn parts and enhance the smooth operation of your wheelchair. To operate properly and safely, your wheelchair must be cared for just like any other vehicle. Routine maintenance will extend the life and efficiency of your wheelchair.

Initial adjustments should be made to suit your personal body structure and preference. Thereafter follow these maintenance procedures:

Safety inspection checklist

⚠ NOTE - Twice a year take your wheelchair to the dealer for a thorough inspection and servicing. Regular cleaning will reveal loose or worn parts and enhance the smooth operation of your wheelchair. To operate properly and safely, your wheelchair must be cared for just like

any other vehicle. Routine maintenance will extend the life and efficiency of your wheelchair. Initial adjustments should be made to suit your personal body structure and preference. Thereafter follow these maintenance procedures:

Item	Initially	Inspect/ adjust weekly	Inspect/ adjust monthly	Inspect/ adjust every 6 months
GENERAL				
The wheelchair rolls straight	X	X		X
Ensure all hardware is tight	X			
WHEEL LOCKS				
⚠ Do not interfere with tires rolling	X		X	
Pivot points are free of wear and slack	X		X	
Wheel locks are firm but easy to engage	X		X	
CLOTHING GUARDS				
Inspect for bent or protruding metal	X			X
Ensure all fasteners are secure	X			X
SEAT/BACK UPHOLSTERY				
Inspect for rips or sagging	X			
Inspect fastening to ensure they are secure	X		X	X
REAR WHEELS				
No excessive side movement or binding when lifted and spun	X			X
Quick-release axles lock properly	X	X		X
FRONT CASTER				
Inspect wheel/fork assembly for proper tension by spinning caster; caster should come to a gradual stop	X		X	
Wheel bearings are clean and free of moisture	X	X		
⚠ CAUTION: Wheels and tires should be checked periodically for cracks and wear, and should be replaced when necessary	X		X	
TIRES				
Inspect for flat spots and wear	X		X	
If pneumatic tires, check for proper installation	X	X		
Inspect rear tires for cracks and wear	X			X
⚠ CAUTION: Wheels and tires should be checked periodically for cracks and wear, and should be replaced when necessary				
CLEANING				
Clean and wax all parts				X
Clean upholstery and armrests				X
Inspect axles are free from dirt, lint, etc			X	
Inspect tilt slides and roller bearings are free from dirt, lint, etc			X	

Troubleshooting

Problems	Solution
Chair veers right Chair veers left Sluggish turn or performance Casters flutter 3 of 4 wheels contacting ground surface	If pneumatic, check tires for correct/equal pressure
Sluggish turn or performance Casters flutter Squeaks and rattles Looseness in chair	Check for loose stem nuts and bolts
Chair veers right Chair veers left Sluggish turn or performance Casters flutter	Check caster angle
Chair veers right Chair veers left Casters flutter	Check that both casters contact ground at the same time

Warranty

The XXL-Rehab Eclipse Tilt is covered by one year warranty from the invoice date for materials and workmanship. The wheelchair frame proper (lower frame) is covered by a five-year warranty from the invoice date.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void. Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

Specifications

Maximum weight capacity is	270 kg
Total width	Seat width + 25 cm
Total length without footrests	87 cm
Seat width	55 / 61 / 66 / 72 / 76 / 82 cm
Seat depth	41-56 cm
Seat height	41-51 cm
Backrest height	65-75 cm
Backrest angel adjustable	Yes
Armrest height adjustable	29-34 cm
Armrests removable	Yes
Footrests height adjustable	Yes
Footrests swing-away and removable	Yes
Weight	Approx. 32 kg
Operation	Manual
Flame-retardant cover	Yes
Color cover	Black
Color frame	Blue
Push handle	Yes
Quick release on rear wheels	Yes
Balance point adjustable	Yes
Fixed seat	Yes
Frame	Powder-lacquered steel
Rear wheels	24" solid PU tires
Casters	125 mm nylon with PU
Seat / backrest	Polyester / nylon
Armrests	Polyurethane
ISO - classification	ISO 12 22 03 - 08

MANUFACTURER

PDG Product Design Group
Unit 103 - 318 East Kent Ave. South
Vancouver, BC, Canada V5X 4N6

DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

03/2019/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com