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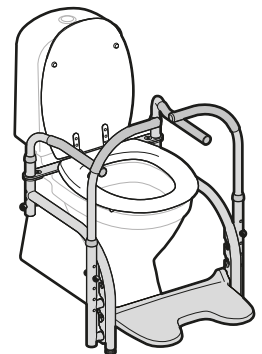
Installation, User Manual and Reconditioning



Read **Installation and User Manual** before install and use of **Balance**. Save manual for future use.

Balance

Item number 10 101 - 10 110

The User Manual refer to batch number:
204288, 204289, 204524 –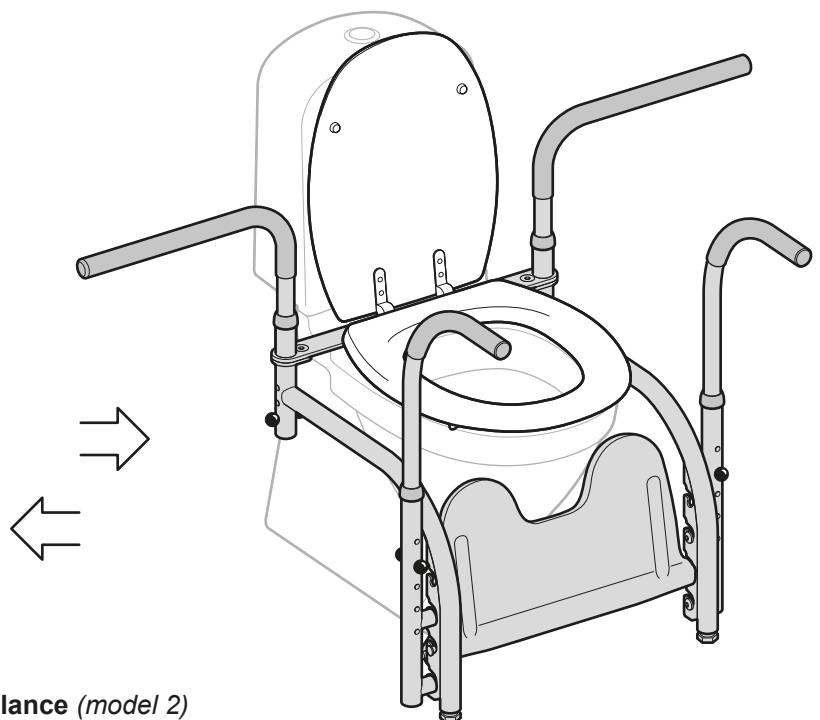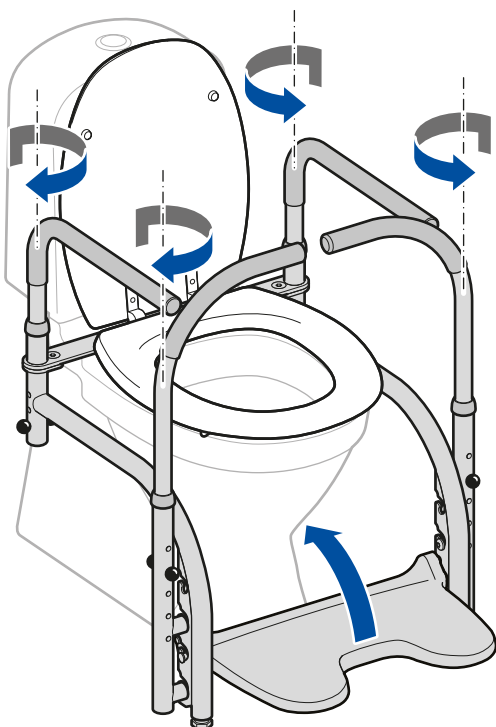

Balance (model 2)

Product description

Balance is an aid for people who requires extra support when moving to and from the Lavatory, and when sitting on the Lavatory.

Balance is a modular system that consists of a numerous of different components that can be combined in various ways.

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
Seiten 13 - 18

Installation, User Manual
and Reconditioning

pages 19 - 24

Exploded view

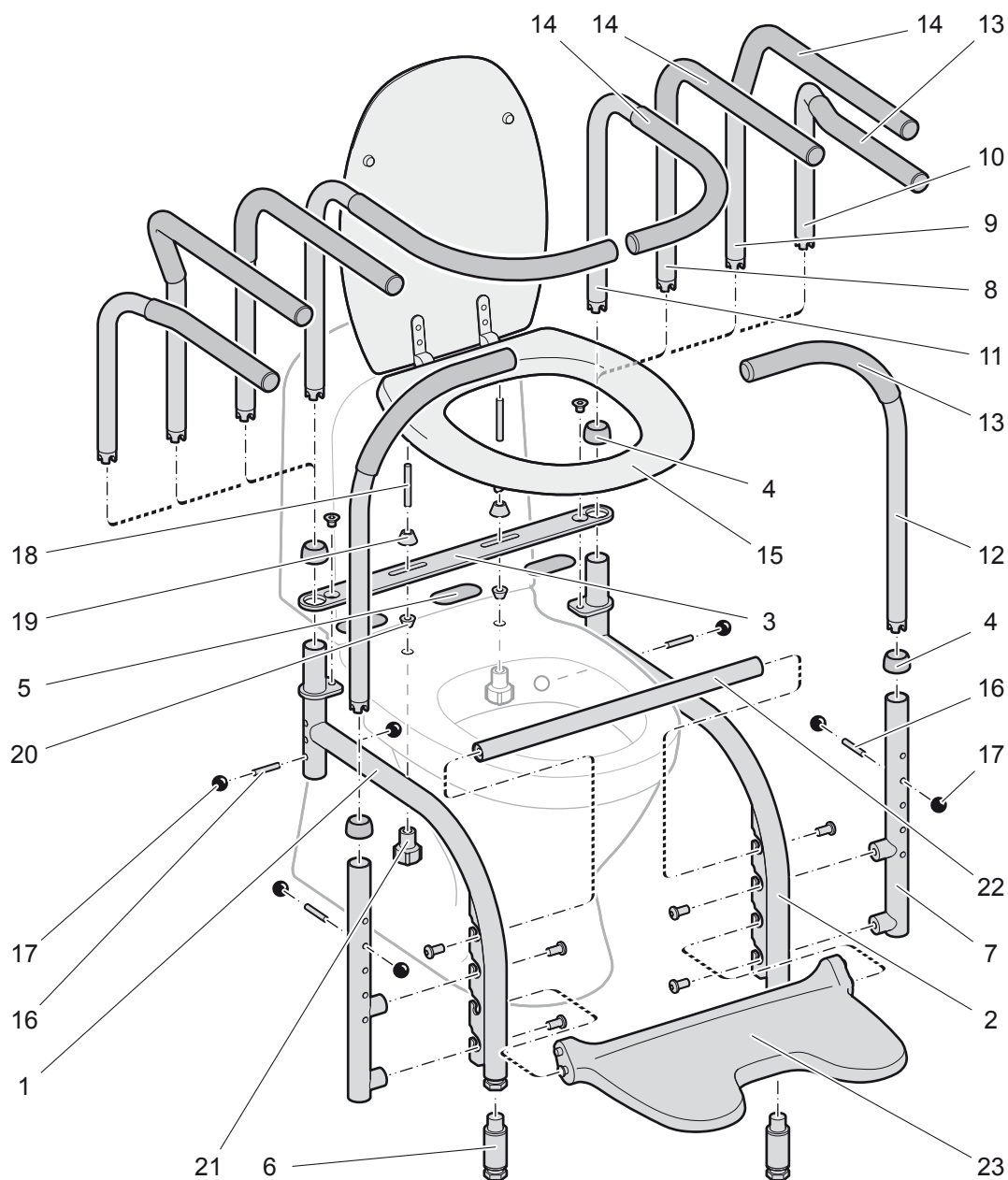Svan Care reserves the right to make changes to the parts list. Current information is available on our website.

Parts list

No	Part	Item number
1	Frame Right (included in Frame, Item no 10 002)	Included in 10 002
2	Frame Left	Included in 10 002
3	Transverse Plate	Included in 10 002
4	Plastic Guard (included in Frame and in Grab Rail Bracket)	Included in 10 002
5	Rubber cloth (36 x 73 mm), piece	10 077
6	Leg extension, pair	10 087
7	Grab Rail Bracket, piece (pos. no. 4, 16, 17 included)	10 031
8	Armrest Straight, pair	10 052
9	Armrest Duo, pair	10 053
10	Armrest Duo Mini, pair	10 084
11	Armrest Surround, pair	10 051
12	Grab Rail, piece	10 091
13	Rubber handle (270 mm)	10 056
14	Rubber handle (440 mm)	10 055
15	Lavatory Seat	10 041

No	Part	Item number
16	Locking pin	10 008
17	Plastic ball	10 009
18	Threaded rod (included in Mounting Kit for Lavatory Seat, Item no 10 099)	Included in 10 099
19	Cone	Included in 10 099
20	Cone for stabilization	Included in 10 099
21	Nut	Included in 10 099
22	Footrest Substitute	10 038
23	Footrest	10 039

How To Use

Before use: Be sure that **Balance** is well mounted and properly secured on the lavatory.

Easy to Clean

Clean the product with a mild detergent and water. If necessary, disinfect.

Reconditioning

Disassemble the entire product. Clean the Lavatory Seat with a non-scratching detergent, rinse with water and dry. Disinfect. Clean the other parts in the same way, alternatively with a dish cabinet or similar equipment. Check all parts for wear and tear, replace if necessary.

Transport and Storage

Always protect parts against external stress during transport.

Material, Environment and Recycling

Balance is made of varnished stainless steel, with details in polyurethane without latex. Lavatory Seat, Handles and Footrest can be turned in for incineration or recycling. Metal parts are to be scrapped.

Specifications

Balance can be combined with a number of products from Svan Care, e.g. Bidette, Hole Reducer and Back Cushion, etc. These products are accompanied by separate instructions.

Important!

Balance may only be installed on lavatories that are firmly anchored to floor or wall.

Never leave a person who is using **Balance** unattended.


In case of any problems with **Balance**, contact an occupational therapist or Svan Care for support.

Svan Care assumes no liability if this product is used, changed or installed in any other way than specified in this Installation and User Manual.

Maximum load: **Armrest 130 kg/pair, Armrest Surround 130 kg/(50 kg)/pair, Grab Rail 130 kg/pair, Frame 130 kg, Footrest 60 kg.**

Mounting of Frame, Lavatory Seat, Grab Rails and Armrests

NOTE! Verify which model you have (different parts are supplied to different models).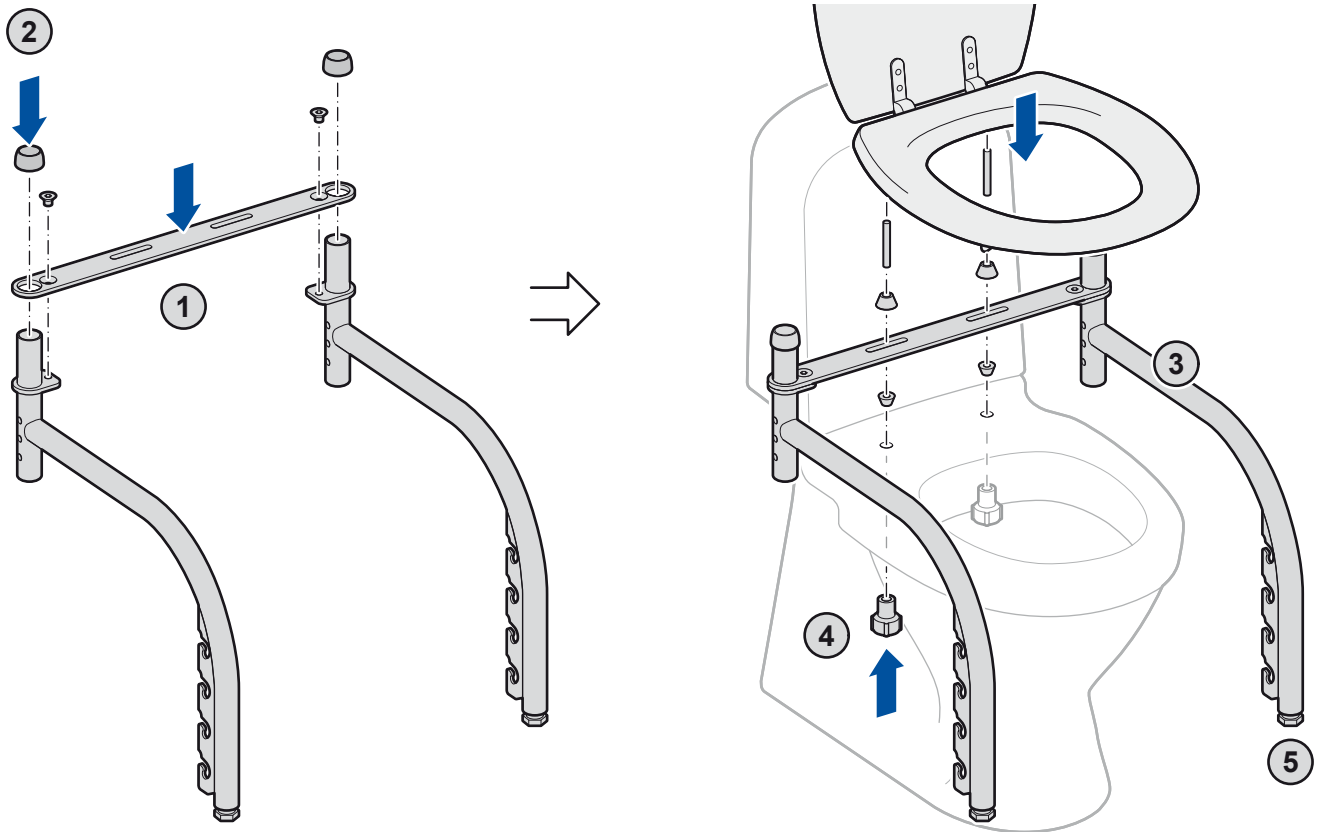

1. Fit the Transverse. Tighten the two countersunk screws with the tool provided.

2. Mount the two Plastic Guards on the Frame.

3. Place the Frame over the Lavatory.


4. Mount the Lavatory Seat. Tighten the two nuts so that the Lavatory Seat holds the Frame on the Lavatory Porcelain. If necessary, use the smaller cone to stabilize.

5. Adjust the Feet of the Frame.


6. Mount the Grab Rail Brackets on the Frame (two options). Tighten the four bolts with the tool provided.

7. Mount the Grab Rails and adjust the height (five options).

8. Mount the Armrests and adjust the height (three options).



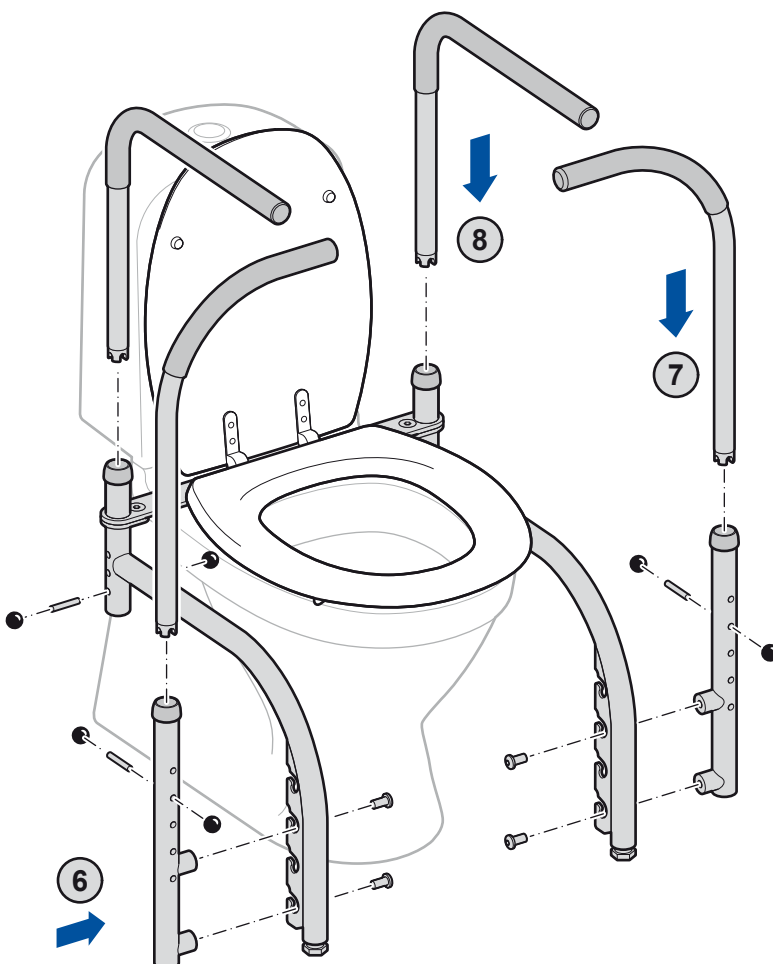



Mounting of Footrest (or Footrest Substitute)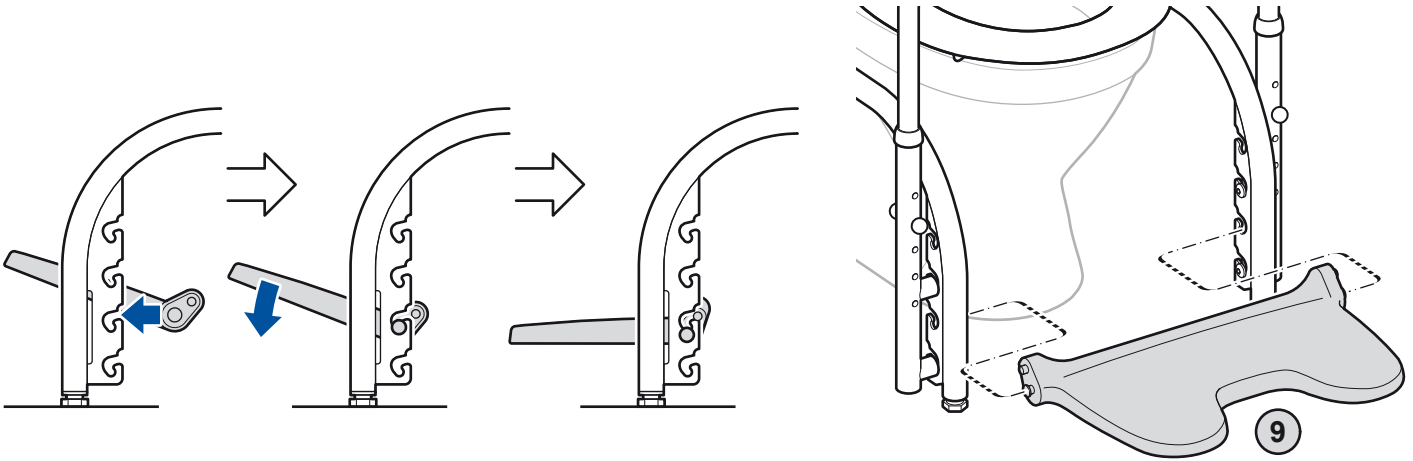


9. Mount the Footrest on a suitable height.

10. If Footrest is not included, mount the Footrest Substitute in the top position. Tighten the two bolts with the tool provided.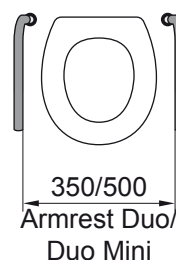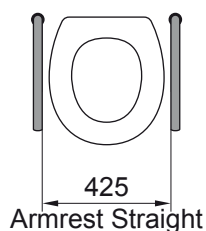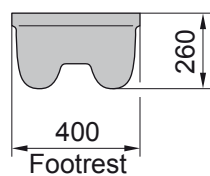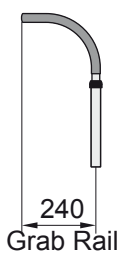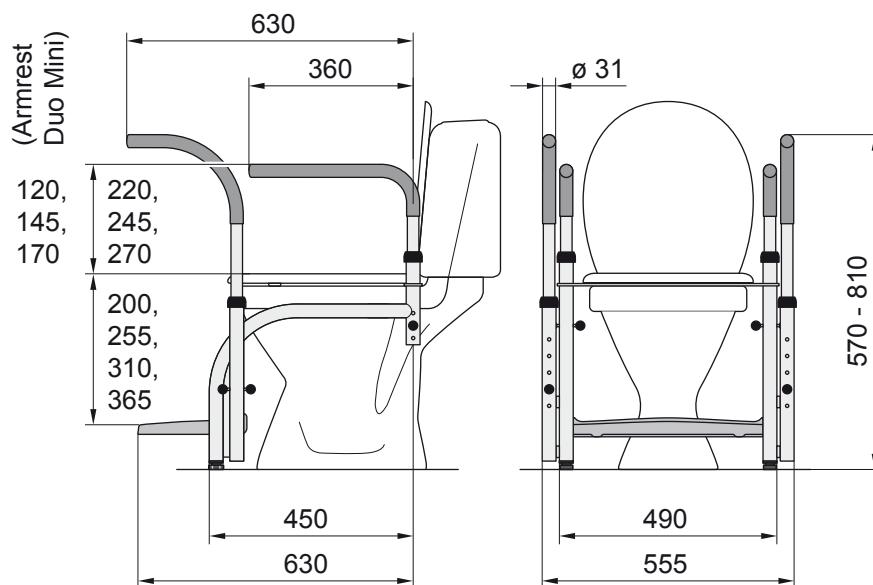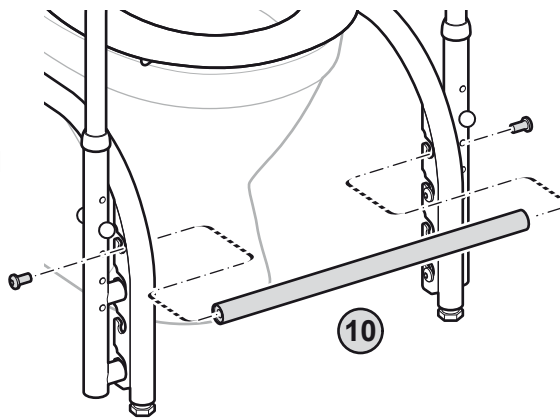


Models of Balance


Model 1: Item no. 10 101
 Armrest Straight, 425 mm
 between the armrests.
 Footrest.
 Frame.
 Lavatory Seat.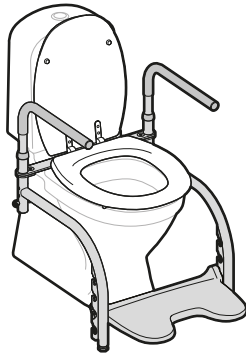


Model 2: Item no. 10 102
 Armrest Straight, 425 mm
 between the armrests.
 Footrest.
 Grab Rails.
 Frame.
 Lavatory Seat.

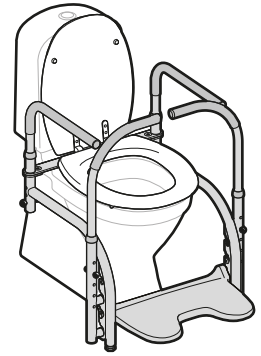
Model 3: Item no. 10 103
 Armrest Straight, 425 mm
 between the armrests.
 Grab Rails.
 Frame.
 Lavatory Seat.
 Footrest Substitute.

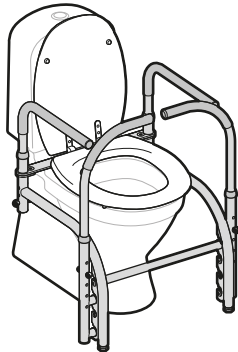
Model 4: Item no. 10 104
 Armrest Duo, 350 alt. 500 mm
 between the armrests.
 Footrest.
 Frame.
 Lavatory Seat.

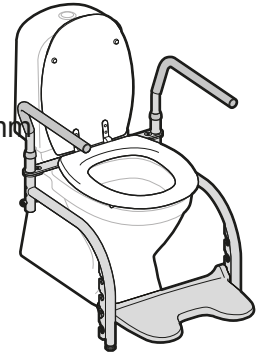
Model 5: Item no. 10 105
 Armrest Duo, 350 alt. 500 mm
 between the armrests.
 Footrest.
 Grab Rails.
 Frame.
 Lavatory Seat.

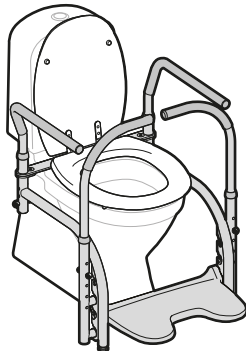
Model 6: Item no. 10 106
 Armrest Duo, 350 alt. 500 mm
 between the armrests.
 Grab Rails.
 Frame.
 Lavatory Seat.
 Footrest Substitute.

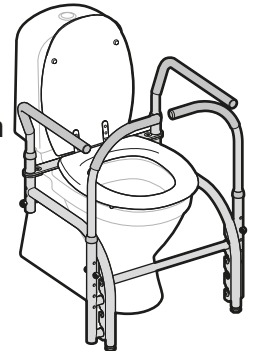
Model 7: Item no. 10 107
 Armrest Surround, 425 mm
 Footrest.
 Frame.
 Lavatory Seat.
 Maximum load/pair: **130 kg**
50 kg

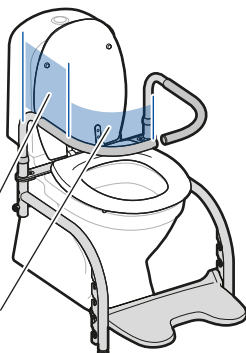Model 8: Item no. 10 108
 Armrest Surround, 425 mm
 Frame.
 Lavatory Seat.
 Footrest Substitute.
 Maximum load/pair: **130 kg**
50 kg

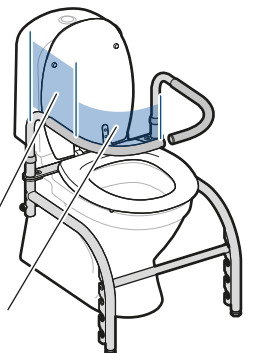Model 9: Item no. 10 109
 Armrest Duo Mini, 350 alt.
 500 mm between the armrests.
 Footrest.
 Frame.
 Lavatory Seat.

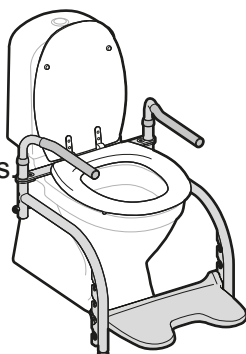Model 10: Item no. 10 110
 Armrest Duo Mini, 350 alt.
 500 mm between the armrests.
 Footrest.
 Grab Rails.
 Frame.
 Lavatory Seat.