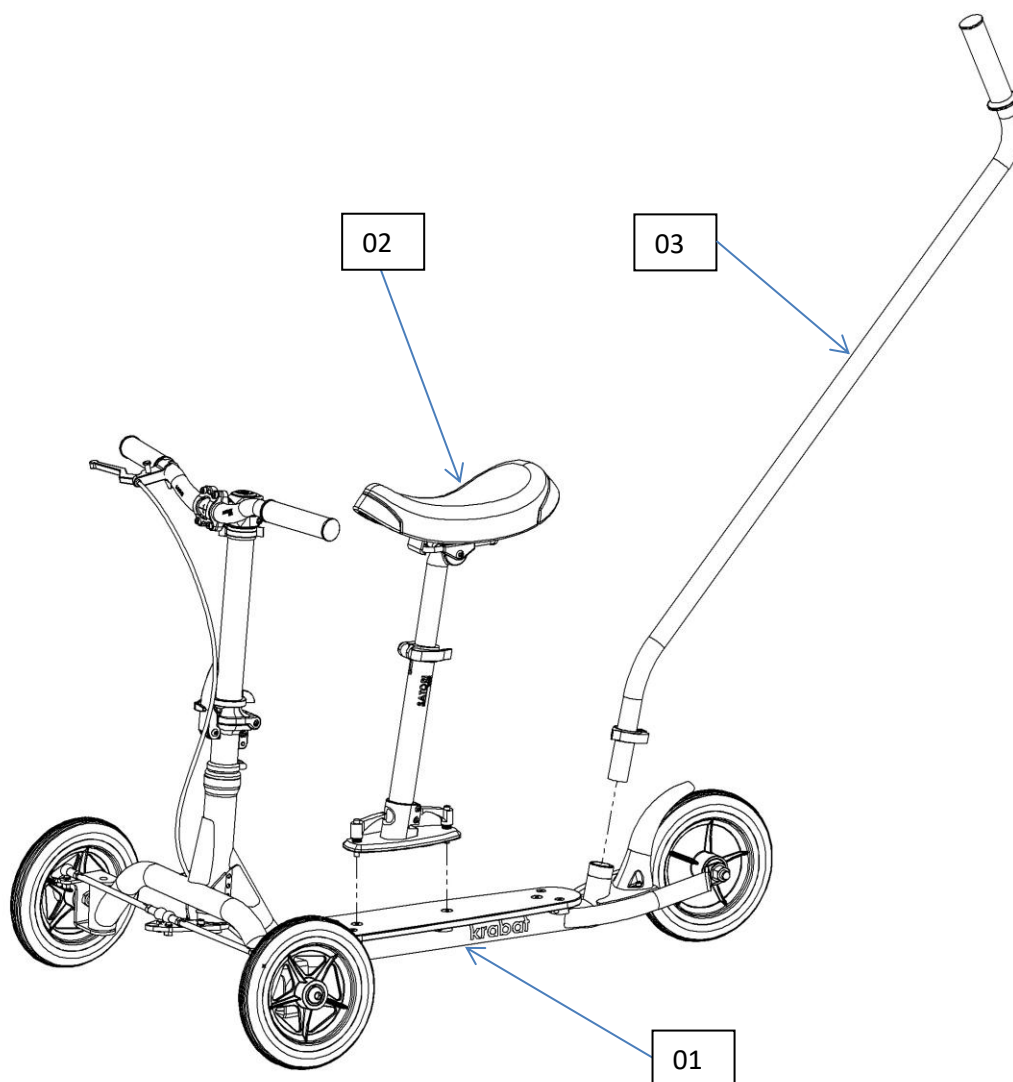
## B-02 – Krabat Runner – Bruksanvisning / User manual

### Hoveddeler/ Main Parts

**01** Krabat Runner

**02** Sete, komplett/ Seat, complete

**03** Ledsagerbøyle, komplett / Push bar, complete

#### Nødvendig verktøy / Necessary tools:

- None

*Note: Dersom setedybde ønskes justert, bruk en 4mm Unbrakonøkkel / If seat depth adjustment is required, use a 4mm Allen key*

**Attention:** Fest skruer og hendler godt / Tighten screws and handles properly

## Monteringsveiledning / Installation instructions:

### Styre/ Handlebar:

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

### Brems - Parkeringsbrems/ Brake - Parking brake:

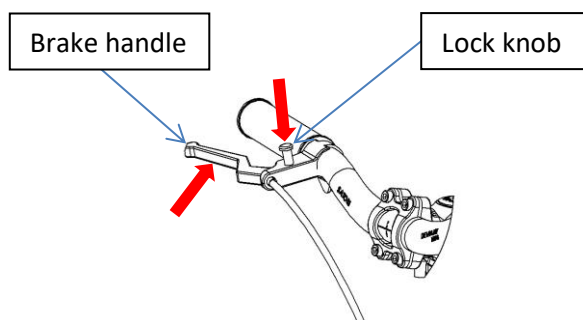Krabat Runner er utstyrt med en kombinert kjørebrems/ parkeringsbrems / Krabat Runner is equipped with a combined brake/ parking brake.

Bremsar bør ha jevnlig vedlikehold for å fungere optimalt / Brakes should be maintained on a regular basis for optimal function

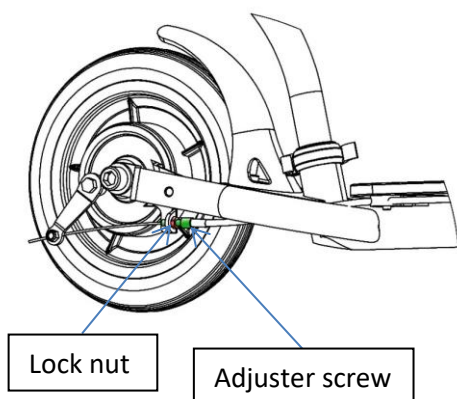10. For å aktivere parkeringsbremsen, press inn bremsehendelen og trykk deretter ned låsekappen / To activate the parking brake, first apply force to the brake handle, then push down the Lock knob

Bremser bør justeres jevnlig for å fungere optimalt /  
Brakes should be adjusted on a regular basis for optimal function

11. For å justere bremsewiren, løsne først låsemutteren og juster deretter bremsewirens stramming med justeringskruen. Stram låsemutteren / To adjust the brake wire, first loosen the lock nut, then adjust the wire using the adjuster screw. Tighten the lock nut.


#### Generelle advarsler/ General attention:

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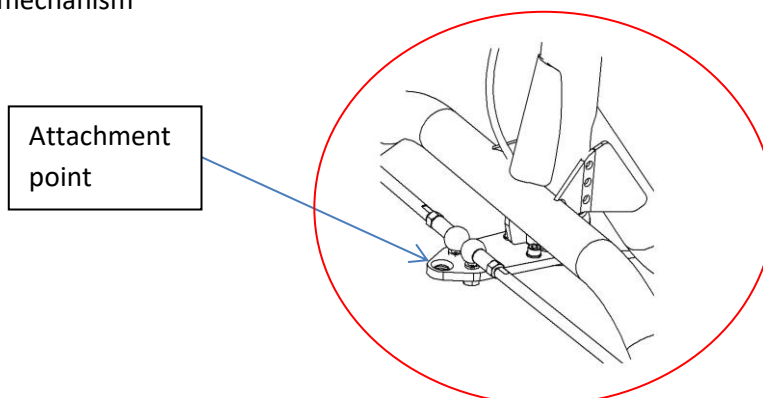## SAFETY

All Krabat products are CE marked according to 93/42 / EØF, Class 1 of 14 June 1993 - Medical Device Directive.

This certifies that it meets all relevant European safety requirements. This manual is approved according to these standards.

The shelf life of the product corresponds to a lifespan of about 5 years under normal use. Then the product should be renovated by Krabat AS or an approved partner to extend the life. All service must be performed by Krabat or an authorized partner.

**Products with electrical components must be submitted to Krabat for a technical review / overhaul before it is sent out to use again.**

The CE-label is to be removed if the product is modified, or non-original parts are used!

Adjustments not described in Krabat user manuals shall only be carried out by Krabat, or by personel approved by Krabat.

Krabat products are not toys! Improper use may cause damage to both user and/ or the product.

## WARRANTY

Krabat provide a guarantee of 2 years on all its products. This guarantee is aimed at defects in workmanship or materials. The guarantee may be reduced or waive if service or daily maintenance are not observed. It is the customer's responsibility to ensure that maintenance is carried out.

The warranty does not cover accidental damage, including damage caused by improper use or lack of maintenance. The warranty does not cover normal wear or wear parts.

The warranty is void if non-original parts, or if the product is repaired or modified by personel not approved by Krabat.